

PK-160 Remote Control



□/田 : button for select full page image or quad images.

Freeze : button for capturing images (same function with footpad capture or capture button on handpiece).

Recall : button for recalling the captured images. After pressing this button, the screen will appear previous captured images start from page 1.

Up & Down : after pressed recall button, press up or down button to get the expected image and press the recall button once again to get the live image.

Delete : button for deleting unwanted images when recalling or previewing images.

NEG : Negatoscope function. Press this button the LCD screen turn to negatoscope screen and user can use it to view large X-Ray film. Press once again it will return to intraoral camera mode.

IS : Image Stabilizer function. After taking intraoral image then press this button, image vibration will be reduced. It also can use to reduce vibration when we "recall" the stored images.

BW : Turn the images into black and white mode.

SAVE : Save the images into SD card.

Delete : Delete images that have stored in SD card one by one.

Setup : Setup Year, Month, Date (YY, MM, DD) in SD card.
Setup time in 24 hours mode.
Format the SD card.

OSD : To display or undisplay (YY, MM, DD)

< > : To preview the images that have been stored in SD card one by one.